English

AH!YES is the USA version of YES products, offering the same great product and excellent customer service, but with a new name especially for the USA!

Please read this leaflet carefully before use.

AH!YESVM Vaginal Moisturizing Gel

AH!YESVM water-based organic vaginal moisturizing gel uses a patented technology to provide natural, long-lasting moisture to dry vaginal mucosa.

Provides symptomatic relief from:

- Vaginal dryness
- Vaginal Atrophy
- Itching
- Irritation, burning and soreness
- Dyspareunia.

Temporary vaginal dryness can have various origins: after childbirth, when breastfeeding, and when taking oral contraceptives. Certain anti-depressants and anti-histamines can cause intimate dryness, as can extreme stress.

Description

- A soft, clear bio-adhesive gel
- Discreet no smell, taste, color, stickiness or fallout. Close to the texture and feel of a woman's own lubrication
- pH matched to restore and stabilize the protective acidity of the intimate environment at pH4, supporting vaginal health.
- Hormone-free
- Not made with any known skin irritants (no glycerine, no hormones, no gluten, no petrochemicals, no parabens).
- Gentle and natural, for even the most sensitive conditions
- Guaranteed natural and pure by its certified organic status
- Recommended by Health Care Professionals and trusted by women to bring soothing, safe and welcome relief to the debilitating symptoms of menopausal dryness
- A non-hormonal natural alternative to synthetic chemical intimate moisturizers.
- Case studies show that AH!YESVM is fast-acting and long-lasting while replenishing intimate moisture safely and naturally.

AH!YESVM is available in 3.4fl oz. tubes and packs of 6 pre-filled applicators.

Directions for use – AH!YESVM in tube

(see overleaf for applicator directions for use)

This product is a soft clear gel, easy to dispense and distribute, and is designed for finger application, which allows you to decide how much you want to put where. Simply squeeze the soft gel on to your hand and use your finger tips to apply the desired amount to the vaginal tissues. AH!YESVM may also be used to moisturize the labia. (No applicator included).

Indications for Use

AH!YESVM vaginal moisturizer is a personal lubricant for vaginal and penile application, intended to moisturize and lubricate, to enhance the comfort and ease of intimate sexual activity and supplement the body's natural lubrication. AH!YESVM vaginal moisturizer is compatible with natural rubber latex and polyisoprene condoms. This product is not compatible with polyurethane condoms.

Ingredients

Aqua, Linum usitatissimum (flax) extract*, Aloe barbadensis leaf juice powder (aloe vera)*, Cyamopsis tetragonolobus (guar gum)*, Ceratonia siliqua (locust bean gum)*, Sodium chloride, Xanthan gum, Potassium sorbate, Citric acid, Phenoxyethanol.

* Organically grown ingredients.

Certified 95% Organic by the Soil Association, Bristol UK

Protected by International Patents.

Storage: Replace cap when not in use. Store in a cool dry place between 5-30°C. Use before expiry date. Keep out of direct sunlight.

Caution: AH!YESVM is not a contraceptive or a spermicide.

Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice.

Why certified organic?

When you buy something that is certified organic, not only are you making the best choice for your own wellbeing, your purchase is also supporting a system that respects human health, animals,

wildlife and the environment. Certified organic products meet stringent standards and organic certification is your assurance that our ingredients and products have been grown and processed according to strict procedures. Production, processing, distribution and sales are monitored to maintain the finest organic and ecological integrity.

Packaging and recycling

AH!YES aims to minimize the environmental impacts of all our packaging. So please do help us achieve that aim. All our bottles are MDPE. Polyethylene (MDPE) is the most widely accepted plastic by recycling centers. The cartons, boxes and package inserts can all be recycled.

100% Guarantee: AH!YES products are 100% satisfaction guaranteed. Please contact us by telephone or email if you have any concerns or questions about this guarantee.

Certified Organic by: The Soil Association, Bristol, UK.

Approved by: The Vegetarian Society.

Manufactured in Great Britain for

Bio-Tech Lubricants Ltd, Contact: The Timber Barn, Selborne Road, Greatham, Liss, Hampshire, GU33 6HG, UK.

AH!YES logos are registered trademarks of YES Syzygy Ltd.

Please contact us by telephone or email if you have any concerns or questions.

Contact us

USA Toll Free: 1-888-777-0125

Website: www.ahyes.org

Email: help@ahyes.org

Our love for Planet Earth is reflected in everything we do – creating glorious products that are amazing to use, kind to the environment, respectful of all life and natural.

Truly, madly, deeply natural.

Sarah and Susi
AH!YES Founders

See our full range overleaf

XAMFUM_V1_02/19

MADE WITH SAFE INGREDIENTS

MADE SAFE

MADE SAFE SCREENS
INGREDIENTS FOR KNOWN:

- ✓ BEHAVIORAL TOXINS
- ✓ CARCINOGENS
- ✓ DEVELOPMENTAL TOXINS
- ✓ ENDOCRINE DISRUPTORS
- ✓ FIRE RETARDANTS
- ✓ HEAVY METALS
- ✓ NEUROTOXINS
- ✓ HIGH RISK PESTICIDES
- ✓ REPRODUCTIVE TOXINS
- ✓ TOXIC SOLVENTS
- ✓ HARMFUL VOCs

It is imperative to follow manufacturer guidelines for use of certified products.

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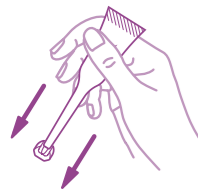
AH!YES[®] VM
Vaginal moisturizing gel


AH!
YES[®]
Natural Intimate Health

Directions for use – AH!YESVM in pre-filled Applicators

When you want to reach higher in the vagina, this product offers you a discreet and convenient way to deliver lubrication and moisture in the right amount to the source of discomfort.

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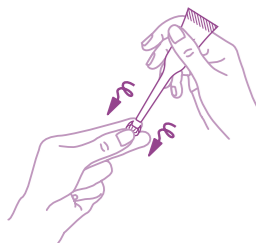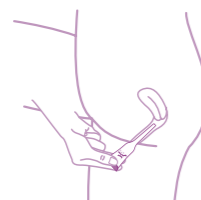
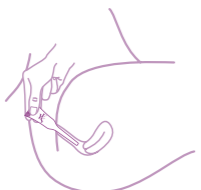
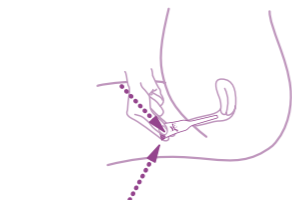
AH!YESVM frequency of use:

A 5ml application used every 2-3 days is typically sufficient to maintain a moist comfortable vagina.

Cases of severe dryness will benefit from more frequent use (can be used daily) initially until improvement is gained.

If using to alleviate painful sex, apply as required before intercourse and to ensure comfort afterwards.

Iso-tonic*. The iso-tonic/iso-osmotic nature of YESVM is carefully matched to human tissues. Why is this balance important? Products which are hyper-tonic (often containing chemicals such as glycerine and glycols) can cause irritation to intimate tissues*. Being iso-tonic/iso-osmotic* YESVM protects against such irritation.

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The AH! YES range

Whether you use a little or a lot, for pleasure or for health, discover the full AH! YES collection of natural and organic intimate products, in a range of sizes and variants to meet your differing needs.

AH!YES WB

Is an intimate lubricant formulated to hydrate and revitalize dry vaginal tissues and alleviate discomfort. AH!YES WB is pH matched to the typical vaginal environment. Maintaining typical vaginal pH may help prevent Thrush and bacterial infections.

AH!YES OB

Is a plant-oil based, longer-lasting lubricant and intense moisturizer, which is soothing and richly emollient. Nourishes delicate intimate tissues. Not condom compatible.

AH!YESVM (Vaginal Moisturiser)

Is a long-lasting vaginal moisturizer. Designed to match the typical vaginal environment with none of the skin damaging or concerning chemicals typically found in vaginal moisturizers.

AH!YES CLEANSE

Is a sensitively formulated intimate wash, designed to respect the delicate vaginal ecology, with a refreshing and effective cleansing action. Available in Unfragranced and Rose.

AH! YES

Natural Intimate Health